

PowerFloat



Catalogue

About Us:

Dennis Rach, was a partner in the 16 veterinarian practice of Moore Equine Veterinary Centre, Calgary, Alberta, Canada He has been in large animal practice with the firm continuously since 1970. As a practitioner, Dr. Rach designed several rotary equine instruments during the 1980's in an attempt to deliver a higher level of dental care to the equine patient. Other equine practitioners were also in need of the same type of equipment so Dr. Rach engaged the services of local machinists to make rotary instruments for other veterinarians.

During the mid 1990's Dr. Rach designed a motorized right angle-shaped floating device. This device worked better than he had expected. The unit stood the durability testing and functioned very well over the next few years so he pursued development and refinement of this new design. A patent was obtained for the new dental instrument and the production and sales of the PowerFloat began with its introduction to veterinarians at the AAEP meeting in Albuquerque, New Mexico in 2000. Over the next 2 years the initial core of veterinarians who purchased the PowerFloat recommended it to other fellow practitioners and the use of the instrument spread throughout the USA and Canada. As well as North America, the PowerFloat began to be sold and serviced through partners in Australia, New Zealand, and Europe.

Since the introduction of the PowerFloat, practitioners throughout the world have helped Dr. Rach make improvements and additions to the basic model. New dental procedures and techniques have been developed and new instrumentation continues to be added to the PowerFloat to deliver safe, superior dental care to the horse.



POWERFLOAT.NET / INFO@POWERFLOAT.NET / PHONE: 1-877-969-2233

VISIT OUR WEBSITE TO SEE OUR FULL LINE OF PRODUCTS.

Order Information:

Visit our Website to order online

PowerFloat.net

Order by Phone

Our Calgary office operates Monday to Friday, 8AM to 4:30PM MST (excluding Canadian holidays). Canadian and USA customers can place an order during business hours through our toll-free number:

1-877-969-2233

Customers outside of North America can place an order by dialing:

+1 403-984-8775

Order by Email

info@powerfloat.net

See us on Facebook and follow us on Instagram.

Warranty Information

PowerFloat Inc. will repair or replace, without charge, any defects due to faulty materials or workmanship for a period of six (6) months from the date of purchase.

This warranty does not cover part failure due to normal wear, failure to follow recommended maintenance procedures or tool abuse.

Repair and Service

Simply visit **powerfloat.net** and download the repair forms for your country and follow their instructions. Otherwise, all customers can call the PowerFloat office toll-free at **1-877-969-2233** for any repair or service assistance during the life of their PowerFloat product.

Quick Connect Models

Plug-in Quick Connect - PF0100



Product Description:

A 120V AC Dewalt® variable speed motor equipped with a proprietary socket that accepts & releases all compatible Quick Connect attachments in seconds.

The Plug-in Quick Connect model operates up to a maximum of 4000 rpm in both Up and Down positions, and comes equipped with hands-free (“locked rpms”) capabilities. Designed to be lightweight and durable to take on the most demanding equine dentistry conditions. This PowerFloat model is only compatible with AC electric sources and must always be operated in conjunction with a portable GFCI (*not included*).

Quick Connect Models

DCD Cordless Quick Connect - PF0210



Product Description:

A variable speed 20V Dewalt® lithium ion motor with Nano™ technology. The DCD cordless PowerFloat model is equipped with a proprietary socket that accepts & releases all compatible Quick Connect attachments in seconds.

The DCD Cordless 20V Quick Connect model operates at up to a maximum of 4000 rpm in the Up and Down positions. Designed for fast, efficient floating of the upper and lower dental arcades in both indoor and outdoor conditions. The DCD comes standard with two large 20V batteries and a 1-hour quick charger for a longer running time.

Quick Connect Models

UltraLite Cordless Quick Connect - PF0300



Product Description:

A variable speed brushless 20V Dewalt® lithium ion motor weighing in at just 3.2 lbs. The UltraLite PowerFloat is equipped with a proprietary socket that accepts & releases all compatible Quick Connect attachments in just seconds.

The UltraLite Cordless Quick Connect model operates at up to a maximum of 3250 rpm in the Up and Down positions. Designed to reduce shoulder and elbow discomfort from longer, intense sessions of equine dentistry. All UltraLite models come standard with two 20V batteries and a 1-hour quick charger.

Quick Connect Models

TL Cordless Quick Connect - PF0200



Product Description:

A variable speed 20V Dewalt® lithium ion motor with Nano™ technology. The cordless PowerFloat motor is equipped with a proprietary socket that accepts & releases all compatible Quick Connect attachments in seconds.

The TL Cordless 20V Quick Connect model operates at up to a maximum of 4000 rpm in the Up and Down positions and comes with hands free (“locking rpms”) capabilities. Designed for fast, efficient floating of the upper and lower dental arcades in both indoor and outdoor conditions. All Cordless models come standard with two 20V batteries and a 1-hour quick charger.

Pre-Configured Kits

Quick Connect: Basic Kit #1



The **Basic Kit #1** is the introduction to the Quick Connect System that allows veterinarians to understand the feel and nuances of the Quick Connect PowerFloat before expanding into more advanced chamfer and rotary attachments.

These kits are designed for veterinarians that require dependable performance today, with the flexibility to add specialized attachments later, as needed. Float dental procedures with ease and unparalleled precision.

The Basic Kit #1 attachments:

Guarded Right Angle with Diamond Grinding Wheel

Available in Plug-in (PF0101), DCD Cordless (PF0211), UltraLite Cordless (PF0301) and TL Cordless (PF0201) models.

***Includes all kit accessory items.**

Quick Connect: Chamfer Kit #2



The **Chamfer Kit #2** is a collection of two of the most popular and heavily-used Quick Connect tools on the market.

Designed to see veterinarians through countless dental procedures, the combination of the *Guarded Right Angle* and the *Un-Guarded Right Angle* can easily handle the majority of advanced equine malocclusions.

The Chamfer Kit #2 attachments:

Guarded Right Angle with Diamond Grinding Wheel

Un-Guarded Right Angle with Chamfer Burr

Available in Plug-in (PF0102), DCD Cordless (PF0212), UltraLite Cordless (PF0302) and TL Cordless (PF0202) models. ***Includes all kit accessory items.**

*** Kit accessory items: Large Carry case, Lg & Regular Gags, Grinding Wheel Removal Tool, Grease Gun, Grease Refill 15 oz, Headlamp, Cleaning Cloth and Scrub Brush.**

POWERFLOAT.NET / INFO@POWERFLOAT.NET / PHONE: 1-877-969-2233

Pre-Configured Kits

Quick Connect: Chamfer & Rotary Kit #3



The **Chamfer & Rotary Kit #3** is a collection of the two most popular and heavily used Quick Connect tools with the addition of the Long rotary shaft to use on the upper and lower 11's.

The Chamfer & Rotary Kit #3 attachments:

Guarded Right Angle with Diamond Grinding Wheel

Un-Guarded Right Angle with Chamfer Burr

Long Rotary with Upper & Lower 11 Burrs OR Long Rotary with Pony Burr and Guard

Available in Plug-in (PF0103) , DCD Cordless (PF0213) , UltraLite Cordless (PF0303) and TL Cordless (PF0203) models.

***Includes all kit accessory items.**

Quick Connect: Complete Kit #4



The **Complete Kit #4** is the complete collection of the most popular and heavily used Quick Connect tools on the market. Switch between these attachments with ease.

The Complete Kit #4 attachments:

Guarded Right Angle with Diamond Grinding Wheel

Un-Guarded Right Angle with Chamfer Burr

Long Rotary with Upper & Lower 11 Burrs OR Long Rotary with Pony Burr and Guard

Short Un-Guarded with Apple Core Burr OR Short Guarded with Diamond Wheel

Available in Plug-in (PF0104), DCD Cordless (PF0214), UltraLite Cordless (PF0304) and TL Cordless (PF0204) models.

***Includes all kit accessory items.**

Quick Connect Attachments

Guarded Right Angle Attachment - PF0403



A long 14" shaft ending in a 90° bend that houses the specially-guarded grinding surface. Quickly connects in the up or down positions for floating of the upper or lower dental arcades. The low-profile design of the right angle allows access to all areas of the dental arcades in horses and some miniature horses. Included standard with all Quick Connect Kits, as well as sold separately.

*Stainless Steel (PF0403) and Aluminum Shafts (PF0403-A) Available.

Un-Guarded Right Angle Attachment - PF0402



A long 14" shaft ending in a slim 90° bend with a un-guarded head. Quickly connects in both left or right horizontal positions and accepts the Chamfer, Apple Core, and other PowerFloat accessory burrs. Our most versatile Quick Connect attachment.

*Stainless Steel (PF0402) and Aluminum Shafts (PF0402-A) Available.

Long Rotary Attachment - Pony - PF0400P



The Long Rotary guarded shaft with a 3/8" carbide burr allows access to all areas of the cheek teeth using a diagonal over-the-tongue approach. This is a must for dental work in all minis and donkeys.

- Compatible with Upper & Lower 11 Burr set (PF0546) - Sold separately

Quick Connect Attachments

Long Rotary Attachment w/Upper & Lower 11 Burrs - PF0400UL



A long 13 inch straight rotary shaft that comes with upper and lower 11 burrs. Quickly connects in both up or down positions. Able to effectively reach the back edge of upper and lower 11 molars.

- Compatible with Pony Rotary Burr Set (PF0545)- Sold separately.

Short Un-Guarded Right Angle Attachment - PF0407



The shortened version of the un-guarded right angle which allows the use of an apple core burr. The apple core burr is introduced into the mouth end on, while the free hand abducts the corner of the mouth to tighten the mandibular frenulum, allowing gentle contouring and smoothing of the sharp edges of the lower front cheek teeth. The upper front cheek teeth are contoured in a similar manner.

Short Guarded Right Angle Attachment - PF0405



The shortened version of the guarded right angle which is used for the reduction of incisor malocclusions. It is designed for veterinarians who wish to have an attachment that allows incisor teeth reductions from a closer range than can be achieved with the longer right angle attachment.

Periotomes

Reciprocating Periotome - PF0408



The PowerFloat Reciprocating Periotome is a specialized right angle shaft that attaches to any of the PowerFloat quick connect drills. The head of the shaft contains a built in specialized collet into which different lengths of 1mm thickness flexible blades are inserted and secured with a set screw. Powered reciprocating strokes cause the blades to penetrate and cut deeper into the periodontal ligament attachments in preparation for cheek teeth extractions. This tool significantly shortens the time required to extract equine cheek teeth.

Rotating Periotome c/w Collet & Rod Set- PF0409



The PowerFloat Rotating Periotome is an unguarded right angle shaft into which a specialized collet and titanium rod set is installed. The rotating periotome is used to drill multiple holes deep into the periodontal space in order to destroy periodontal tissue during extraction procedures. This tool significantly shortens the time required to extract equine cheek teeth.

Dental Halter

Dental Halter - PF0600

Heavy duty webbing and hardware make this halter very durable. Rope and cleat system attached to the nose band allows the veterinarian to adjust the horse's head to a comfortable position. The chin support is made from a high molecular weight composite, which is covered with reinforced tubing.



* Comes in Horse (PF0600) and Pony (PF0601)sizes.

Rope and Cleat Set PF0602



Rope Only PF0604



Cleat Only PF0603



The adjustable rope and cleat system is attached to the nose band of the PowerFloat dental halter. The cleat is made from heavy duty composite and can support a load of 2000 Kg. Flip out action releases the rope from the locked position. Very safe to use due to the quick release system.


Incisor Gags - 3 sizes

Reinforced mouth gags which can be placed in the interdental space of the horse's mouth.



Mouth gags allow the mouth to be held open so that dentistry work can be done on incisors or canine teeth. Knots in the elastic rope can be easily re-adjusted to create a custom fit for all sizes. Available in large (PF0704), regular (PF0703) and pony sized (PF0712 - not shown).

Grinding Wheels & Burrs

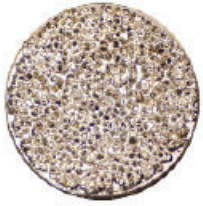

Diamond Grinding Wheel - PF0500

The wheel of choice among veterinarians and the highest performing grinding wheel available for PowerFloats. Runs smoothly and quietly with minimal vibration.

*Threads onto the Guarded Right Angle attachment.



Solid Grinding Wheel - PF0502

The Solid Carbide wheel has a cut pattern that is designed for the rapid reduction of dominant teeth. Its rounded radius design assists in keeping the cutting surface from contacting soft tissue.

*Threads onto the Guarded Right Angle attachment.



Upper 11 Burr - PF0505

Designed to smooth the caudal sharp edges and hooks on the last upper molars (111 / 211).

*Threads onto the Long Rotary attachment.



Lower 11 Burr - PF0506

Shaped to smooth the caudal sharp edges and hooks on the last lower molars (311 / 411).

*Threads onto the Long Rotary attachment.

Grinding Wheel & Burrs

Chamfer Burr - PF0503



Enables reduction of sharp enamel points on arcades at variations of 45°.

Two sizes available:

PF0503 - Regular Chamfer Burr

PF0504 - Small Chamfer Burr

*Threads onto the Un-Guarded Right Angle attachment.



Apple Core Diamond Burr

Used to contour upper and lower front teeth (06s), and canine teeth.

Three models available.

PF0512 - Small Apple Core Burr

PF0513 - Large Apple Core Burr

PF0513-F - Flat Ended Apple Core Burr

*Threads onto the Short and Long attachments.



Diastema Burrs Set of 4 - PF0511

The set of carbide diastema burrs are graduated in shape and width to allow gradual widening of various types of diastema primarily seen in geriatric horses.

Tapered, Conical, Slimline and Periodontal Burr

Dental Speculums

PowerFloat Speculum (screw type) - PF0702



Heavy-duty speculum has two (2) threaded tightening knobs providing smooth, easy opening of the incisor bite plates. Extra wide incisor plates prevent the teeth from slipping off the bite plate.

Has a large side port for easy access to the corners of the mouth. The side arm connections to the incisor plates are machined in such a way to prevent side arm pressure on the horse's cheek.

- Bi-Directional: can be positioned with tightening knobs up or down.
- Arrow indicator on the side arm pivot indicates when optimum opening width for most horses has been reached (*3.5 inches, or 9 cm wide*).

Regular Bite Plates - PF0714



Dental Speculums

McPherson Speculum (ratchet-type) - PF0700



The stainless steel McPherson style speculum is an essential accessory for equine dentistry. This easy to operate speculum is made of high quality, stainless steel construction and is suitable for horses of all breeds and sizes. Also available as a pony model for miniature horses, donkeys and small ponies.

Durable biothane straps are long lasting. The side arms are designed so that they do not close into the horse's cheek (spreader effect). The ratchet mechanism is designed for easier opening than all other McPherson style speculums on the market. Six dependable ratchet notches for easier, smaller, incremental opening. The average opening range of 3 1/2" (8.90 cm) is set on the 5th notch. This graduated feature makes this speculum much easier to open than other speculums of similar design.

Wide Bite Plates - PF0715



Irrigation/Perio Systems

PowerSpray Irrigation System - PF2000



The PowerSpray Irrigation system is a comprehensive system with two misting nozzles that deliver a fine mist to eliminate heat production from grinding surface and reduce the inhalation of tooth dust. In addition a high pressure right angle periodontal pistol and spray wand are included for cleaning periodontal pockets on the lingual or buccal sides of the teeth.

Comes complete with a 3L collapsible water reservoir. The system is powered by a 12 V Lithium Iron Phosphate rechargeable battery and a 150 psi diaphragm pump.

PowerPerio Complete System - PF2001

The PowerPerio Complete System delivers a high pressure spray through a right angle nozzle which fits into periodontal pockets on the lingual or buccal sides of the teeth. The system features a quick disconnect periodontal pistol and spray wand and a 3L collapsible water reservoir.

The system is powered by a 12 V Lithium Iron Phosphate rechargeable battery and a 150 psi diaphragm pump.



Irrigation/Perio Systems

PowerPerio Basic System - PF2002

Heavy duty pistol with quick connect stainless steel spray wand. The microspray orifice of 0.7mm diameter outputs the most efficient pressure spray pattern from a water line. Stable and clinic water line pressures tend to range from 35 psi to 80 psi. An economical way to blow food material out from the majority of equine periodontal pockets. (Hose not included)



PowerCool Sectioning Pump With or Without Perio Wand - PF2010/PF2011



The PowerCool Sectioning Pump is an economical water system for occasional advanced dentistry. The system comes with one spray nozzle that delivers a fine water spray which eliminates heat production from grinding surface and reduces inhalation of tooth dust. Optional quick disconnect stainless steel perio spray wand available. Features a 3.8L refillable hand pump.

Sectioning Burr Set - PF0544



The PowerFloat Sectioning Burr Set will allow the veterinarian to drill pilot holes into teeth, intraoral tooth sectioning, fragment removal, occlusal relief cutting for treatment of periodontal pockets and diastema widening. The set contains a collet designed to be inserted into either the PoweFloat guarded or un-guarded right angle attachments. Included in the set are a variety of burs and bits that are secured into the collet using a thread locking screw set.

PowerLite 5



COMING SOON ! The New PowerLite 5

PowerLite 5 - PF0800

PowerLite 5 is a fully portable, battery-operated light source designed to attach to the upper incisor plate of an equine dental speculum. Its compact, water-resistant design makes this product the most reliable and convenient light source available to practicing veterinarian's.

The light can be concentrated on any area of the mouth by simply changing positions.

PowerLite 5 eliminates the need for cumbersome head-mounted or handheld lights, giving you a freely mobile, high-intensity light source to clearly illuminate your workspace without significantly reducing the opening between the incisor plates. A tool such as this also aids veterinarians in educating horse owners about the benefits of equine dentistry.

Extractors & Forceps

Cap Extractor w/ Speculum Opener - PF0903



Reynolds-style, 16 inch (40.6 cm) cap extractor, with machined grooves in the extractor head for use as a mouth speculum opener. The cap extractor's jaws can be used to pry open full mouth speculums in situations where the horse is biting down onto the incisor plates hard enough that the speculum is difficult to open by hand.

Long Nosed Fragment Forcep- Spring Action - PF0925



The Long Nosed Fragment forcep have long narrow jaws that allow access to many tooth fragments which are not accessible with most fragment forceps. The long nosed design gives extra grasping strength of tooth fragments along with the serrated, 2" long jaws. 16" Long. Stainless Steel construction.

Concave Cap - Geriatric Forcep 17" - PF0914



Low-profile concave design is excellent for grasping and extraction of retained caps, sagittal fracture fragments, and geriatric teeth. This forcep is one of the most versatile extraction forceps that a veterinarian can have for routine dentistry.

Four Prong Extraction Forcep 19" - Horse -



Horse Size: The low-profile four (4) prong forcep fits onto a wider variety of cheek teeth than any other design on the market. It's heavy duty design makes it the forcep of choice for a large percentage of upper and lower cheek teeth extractions.

Extractors & Forceps

Serrated Lower Jaw Molar Extraction Forcep -PF0937



The Serrated Jaw Lower Molar Extraction Forceps are used to assist and provide leverage in the extraction of molars.

PowerFloat Incisor Extraction Kit - PF0937



The PowerFloat incisor extraction kit is used for extracting and loosening equine incisors, as well as wolf teeth and fragments. Kit comes with six instruments and a wallet for safe keeping and protection. Stainless Steel Construction.

Offset Forcep 8" - PF0926



Stainless steel wolf tooth forcep. Concave jaws are excellent for grasping a variety of sizes and shapes of wolf teeth.

Wolf Tooth Forcep 9 1/2" and 11"



Stainless steel wolf tooth forcep. Broad-knurled handles provide excellent grip. Concave jaws are excellent for grasping a variety of sizes and shapes of equine wolf teeth.

Comes in 2 sizes: PF0935 - Wolf Tooth Forcep 9 1/2" and PF0906 - Wolf Tooth Forcep 11"

Elevators/Explorers


PowerFloat Wolf Tooth Elevator Set with Percussion - PF0940

This set includes 4 elevator attachments to fit all sizes of wolf teeth. The slightly concave shaped, thin, sharp elevator tips are designed to prevent the elevator from slipping off the tooth interface during vertical percussion. Then gently elevate the wolf tooth from its socket. The PowerFloat percussion system greatly increases the speed and ease of successful wolf tooth extractions.



PowerFloat Wolf Tooth Elevator Set - PF0936

Handle with non-slip, cutting/elevator design makes wolf tooth extractions easy. After local anesthesia, the elevators are secured on the caudal aspect of the wolf tooth. A hammer is used to gently percuss the cutting edge up into the ligamentous attachment. The long offset handle gives adequate leverage to elevate wolf teeth.

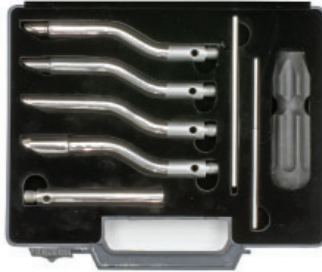

Burgess Hammer - PF0924

Stainless Steel Dental Hammer, used with modified wolf tooth elevator set or incisor extraction set.



Elevators

Periosteal Elevator Set of 3 - PF0931

Periodontal Elevator OR Root Elevator Set:

Set of 3 T-handles allows a veterinarian to choose rotational placement and working height of sizes small, medium and large curved elevators. The elevators can be positioned at a variety of up and down angled positions.



Wolf Tooth Elevator 8" - PF0909



Stainless steel wolf tooth elevator with a bulb handle. This elevator is used to elevate the wolf tooth periodontal ligament without the use of a speculum or via the side port of a full mouth speculum.

Wolf Tooth Elevator 12" - PF0910



Stainless steel wolf tooth elevator with a bulb handle. This elevator is long enough to access wolf teeth with the full mouth speculum in place.

Spreaders

Spreaders (3 sizes)

Stainless Steel Molar Spreaders designed to close at 90° angles to the interproximal space. Comes in 3 sizes. Necessary instrument for complex molar extractions.

3mm Thin - PF0900



5mm Medium - PF0901



6mm Thick - PF0902



Explorers/Drenching Syringe

Dental Explorer 13" - PF0929

Equine flexible taper point Stainless steel - For Ponies



Dental Explorer 15" - PF0930

Equine flexible taper point Stainless steel - For Light horses



Dental Explorer 18" - PF0928

Equine flexible taper point Stainless steel - For Large Horses




Dosing/Drenching Syringe - PF1215

Dosing / Drenching Syringe with two stainless steel nozzles. Capacity of 400cc which is ideal for use to clear hay or food particles from the oral cavity prior to equine dental procedures. Manufactured with an almost unbreakable clear nylon barrel.



Hand Tools

Dental Mirror - PF1002

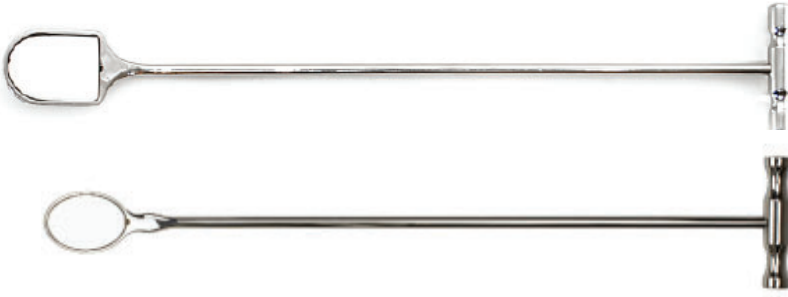

Stainless steel Dental Mirror designed for optimal viewing of all intra-oral structures including occlusal surfaces, periodontal pockets, and diastemata. 18.5 inch (47 cm) stainless steel dental mirror with convenient "T" handle. The 10° tilted endpiece contains a 2 inch (5 cm) diameter mirror. (*Replacement mirrors available*)
*Also available in a smaller oval (PF1006) design.

Dental Probe - PF1001



Stainless steel dental probe with 18 inch (45 cm) handle and 1.5 inch (3.8 mm) hook-style end with marker increments every 5 mm. Knurled handle for better grip. Designed for examination of periodontal attachments to equine cheek teeth.

Dental Pick - PF1000



18 inch (46 cm) stainless steel dental pick with T-handle for easy manipulation in the caudal aspect of the horse's mouth. Pick end height is 1.45 inch (38 mm). Right angle design for easy access to deep areas at the caudal aspect of the mouth .

Accessories

Grinding Wheel Removal Tool - PF1203

Custom-designed hand-tool built to precisely match PowerFloat grinding wheels for quick and easy removal. Tightly grip a wheel's edges without adding unnecessary wear. Grip handles allow for greater leverage when removing stubborn wheels. The best tool available to remove grinding wheels.



Grease Gun - PF1201

6.5 inch (16.5 cm) long taper point grease gun. The tapered end fits into the 2 grease ports located on the right angle of the PowerFloat. The grease is injected into the grease ports with force through a plunger effect.



15 oz Lubricant For Grease Gun - PF1202

Premium food machinery approved grease. Outstanding corrosion and rust resistance. Excellent water resistance which is very important to alleviate the effects of moisture exposure to the gears.



Carry Case - NANUK 940® - PF1200

All of our kits are packaged in the Nanuk® 940 Case.

Waterproof, dustproof, impenetrable and indestructible NK-7 resign shell. Soft grip and ergonomic handle. Large but lightweight with PowerClaw superior latching system. Automatic pressure release valve. Backed by a lifetime warranty.



Ponies, Minis & Donkeys

Dental Halter - Pony - PF0701



This pony sized dental halter comes with the heavy duty webbing and hardware that make this halter very durable. Rope and cleat system attached to the nose band allows the veterinarian to adjust the ponies head to a comfortable position. The chin support is made from high molecular weight composite, which is covered

Dental Speculum - Pony - PF0701



The stainless steel McPherson Pony Style Speculum is an essential accessory for working with miniature horses, small ponies and donkeys.

This smaller design has all the features of the horse size speculum. This includes side arms that are designed so that they do not close into the ponies cheek. Six dependable ratchet notches for easier, smaller incremental opening.

Incisor Gag - Pony - PF0712



This pony sized mouth gag allows the mouth to be held open so that dentistry work can be done on incisors and canine teeth in ponies and minis. Elastic rope can be adjusted to create a custom fit.

Additional attachments for Ponies, Minis & Donkeys:

Short Guarded Right Angle with Diamond Grinding Wheel (PF0405)
Short Un-guarded Right Angle with small Apple Core Burr (PF0407)

Ponies, Minis & Donkeys

Chamfer Burr - Small - PF0504



This pony size chamfer burr enables reduction of sharp enamel points on arcades at variations of 45°.

Diamond Apple Core - Small - PF0512



Used to contour upper and lower front teeth (06s), and canine teeth.

Four Prong Extraction Forcep 19" - Pony - PF0908



Pony Size: A low-profile design and smaller size makes this four (4) prong forcep the best overall instrument to extract cheek teeth in small ponies and miniature horses.

Long Rotary Attachment - Pony - PF0400P



This Long Rotary guarded shaft with a 3/8" carbide burr allows access to all areas of the cheek teeth using a diagonal over-the-tongue approach. This is a must for dental work in all minis and donkeys.

- Compatible with Upper & Lower 11 Burr set (PF0546) - Sold separately



**Phone Toll Free:
1-877-969-2233**

**PowerFloat Inc.
#112, 7865 - 56th Street SE
CALGARY, AB T2C 5S7
CANADA**